POD K8 CARBON GINOCCHIERE

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X-Large: larghezza ginocchio 12,0 – 12,7 cm

Sizing Guide:

POD knee braces are size and side specific and correct sizing is vital to the performance of the brace. Due to varying shapes, knee measurements should only be used as a guide when determining the correct brace size.

KNEE MEASUREMENT

KNEE WIDTH

Knee width measurement should be taken across the centre of the knee using the POD® Sizing Tool. Take measurements in a weight bearing position at full extension (i.e. with leg fully straight).

Size	Knee Width Measurements
Youth Medium	8.8 - 9.4 cm 3.5 - 3.7"
Youth Large	9.4 - 10.2 cm 3.7 - 4.0"
Small	10.2 - 10.8 cm 4.0 - 4.3"
Medium	10.8 - 11.4 cm 4.3 - 4.5"
Large	11.4 - 12.0 cm 4.5 - 4.7"
X-Large	12 - 12.7 cm 4.7 - 5.0"

KNEE CIRCUMFERENCE

If you don't have access to a sizing tool a measurement can be taken of the knee circumference. Knee circumference measurement should be taken directly under the knee cap using a soft tape measure or string. Take measurements in a weight bearing position at full extension (i.e. with leg fully straight).

Size	Knee Circumference Measurements
Youth Medium	27.5 - 29.5 cm 10.8 - 11.6"
Youth Large	29.5 - 32 cm 11.6 - 12.6"
Small	32 - 34 cm 12.6 - 13.4"
Medium	34 - 36 cm 13.4 - 14.2"
Large	36 - 38 cm 14.2 - 15.0"
X-Large	38 - 40.2 cm 15.0 - 15.8"

NOTE

Please be aware that POD® Knee Braces are size and side specific. Brace function will be compromised if the incorrect size and/or side is fitted

STEP 1: Knee Width

Measure across the centre of the knee using the POD Sizing Tool. Take measurements in a weight bearing position at full extension (i.e. with leg fully straight).

STEP 2: Knee Circumference Measurement

Measure the circumference of the knee directly underneath the kneecap using a soft tape measure or string. Take measurements in a weight bearing position at full extension (i.e. with leg fully straight).

STEP 3: Try Brace On

Walk and bend your leg to aid the brace in settling. The brace should feel comfortably snug.

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